
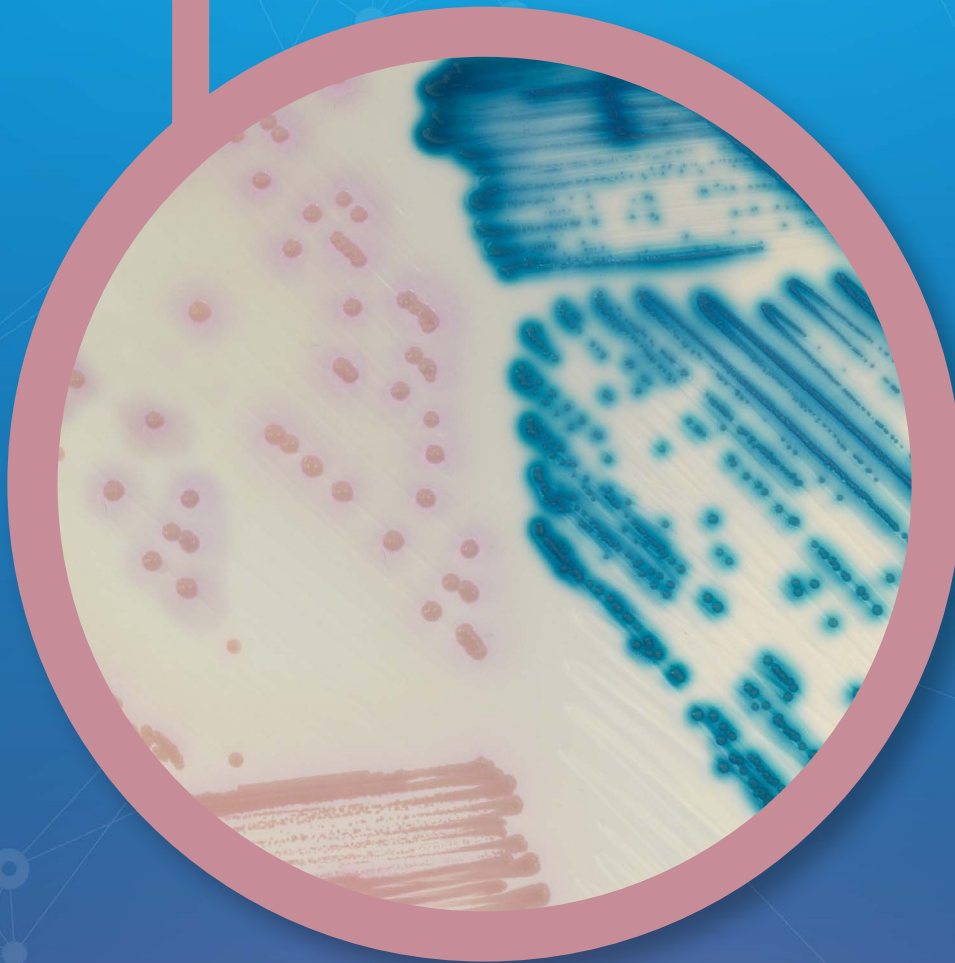
CHROMagar™ LIN-R

Click below:

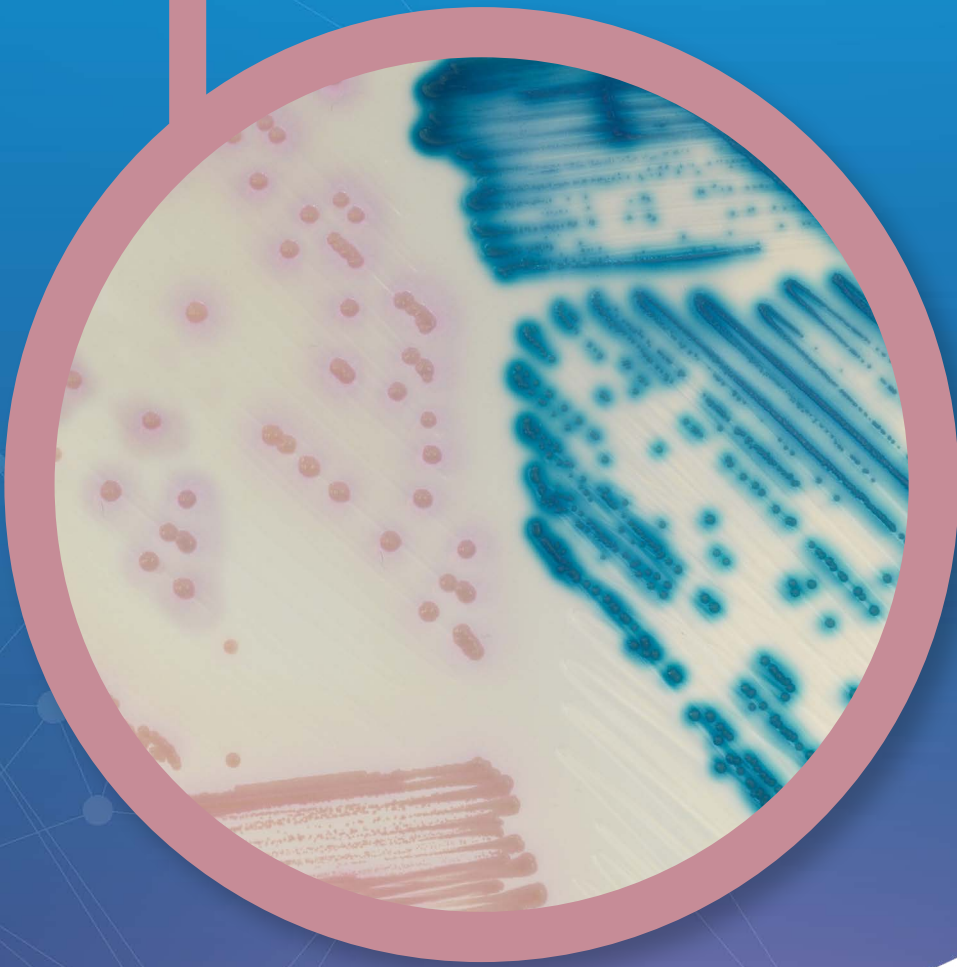CHROMagar™ LIN-R

**For detection and differentiation of
Gram (+) bacteria resistant to linezolid**



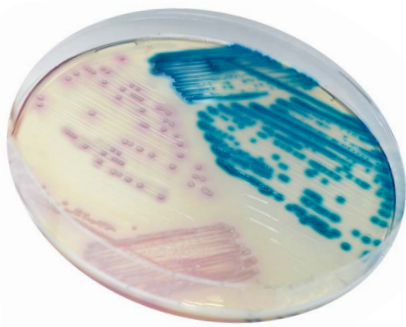
Plate Reading

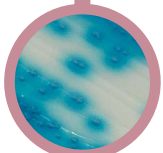• LZD^R *Enterococcus*
→ Steel blue

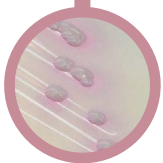• LZD^R *S. aureus*,
S. epidermidis
→ Pink

For detection and differentiation of Gram (+) bacteria resistant to linezolid

Background

Gram (+) cocci pose a global threat to human health due to the emergence of resistance to antibiotics. Linezolid has a broad spectrum of activity against a variety of pathogenic Gram (+) microorganisms, such as MRSA, VRS, and VRE. However, the emergence of strains resistant to linezolid (LIN-R) and horizontal spread of resistance linked to the *cfr* gene, have been increasingly reported after approval of its clinical use.

Although the prevalence of linezolid resistance remains low, the emergence of LIN-R strains is still of great concern. Today, linezolid sensitivity in Gram (+) clinical specimens is primarily monitored by surveillance programs in Europe and in the United States. Clinical isolates for surveillance of LIN-R strains include swabs from the nose (for screening of *Staphylococcus*), perianal and rectal areas (for screening of *Enterococcus*).

CHROMagar™ LIN-R is a chromogenic screening medium for the detection, isolation and differentiation of strains of *Staphylococcus* and *Enterococcus* resistant to linezolid.

Medium Performance

- 1 **HIGHLY SENSITIVE**
Detection of MIC as low as 8 µg/mL.
- 2 **DIRECTLY FROM SPECIMEN**
- 3 **EASY INTERPRETATION**
Species identification by MALDI-TOF can be carried directly for a colony.
- 4 With experience, a trained eye can differentiate between *E. faecium* and *E. faecalis*, and between *S. aureus* and *S. epidermidis*.

Medium Description

Powder Base	Total	42.4 g/L
	Agar	15.0
	Peptones	20.0
	Salt	7.0
	Chromogenic and selective mix	0.4
	Storage at 15/30 °C - pH: 6.9 ± 0.2.	
	Shelf Life	> 18 months
Supplement (Included in the pack)	Liquid form	8 mL/L
	Storage at 15/30 °C Shelf Life	> 18 months

Usual Samples	Stools and nasal, rectal and perianal swabs.
Procedure	Direct Streaking. Incubation 36-48 h, 35-37 °C. Aerobic conditions.

Scientific Publications on this product: available on www.CHROMagar.com
Please read carefully the instructions for use (IFU document) available on www.CHROMagar.com

Order References

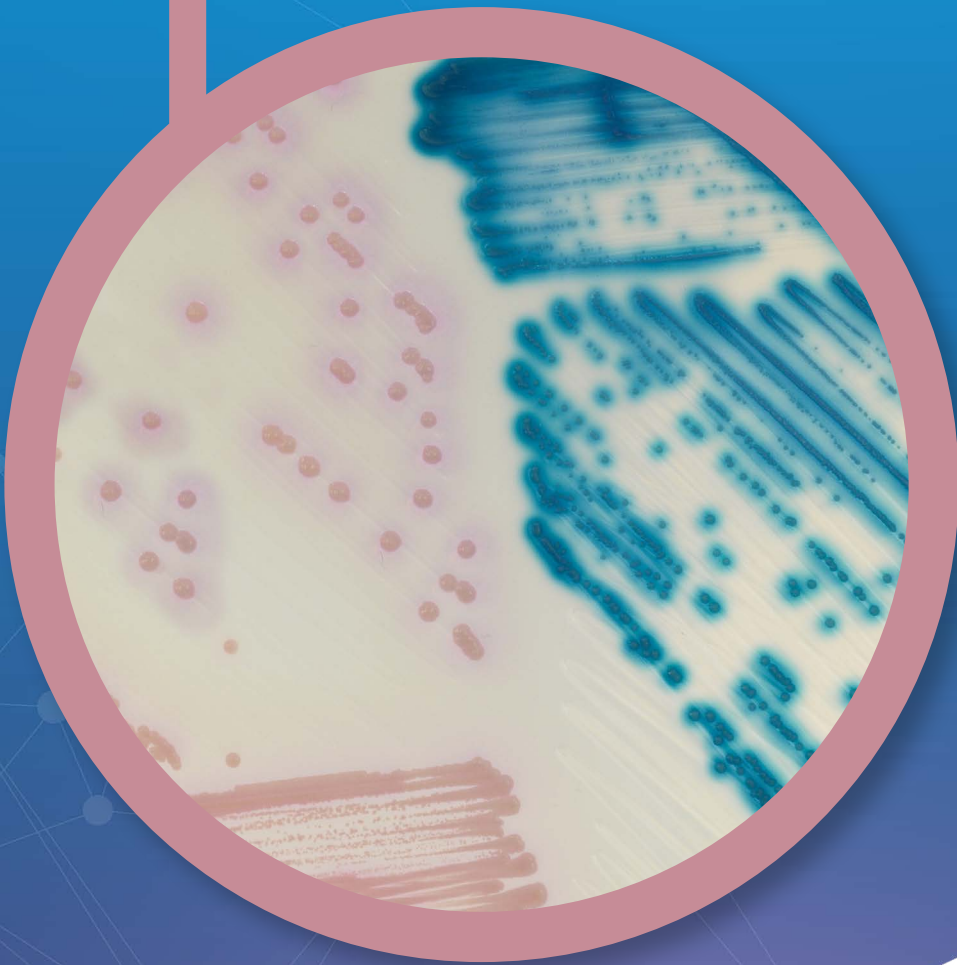
Please use these product references when contacting your local distributor:

5000 mL pack LN762 (included in this reference: powder base LN762(B) + suppl. LN762(S))

Manufacturer: CHROMagar, 29 avenue George Sand,
93210 La Plaine Saint-Denis - France
Email: CHROMagar@CHROMagar.com
Website: www.CHROMagar.com

Find your nearest distributor on
www.CHROMagar.com/contact

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