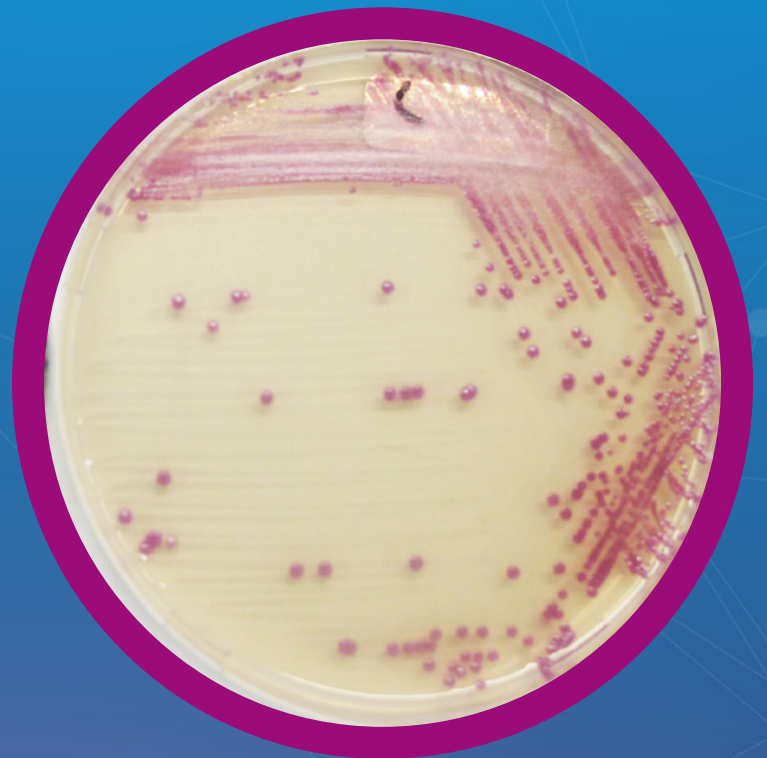
LIM RambaQUICK™ StrepB Method

+

Click below:

[EN](#)

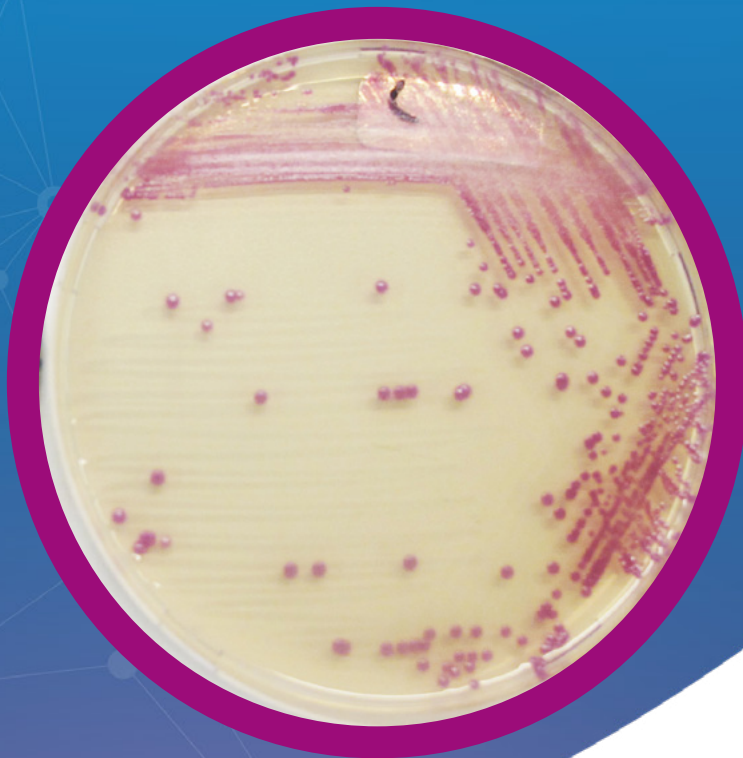
[FR](#)

[ES](#)

LIM RambaQUICK™ StrepB Method

+

For Group B *Streptococcus* (GBS) screening

LIM RambaQUICK™ StrepB Method

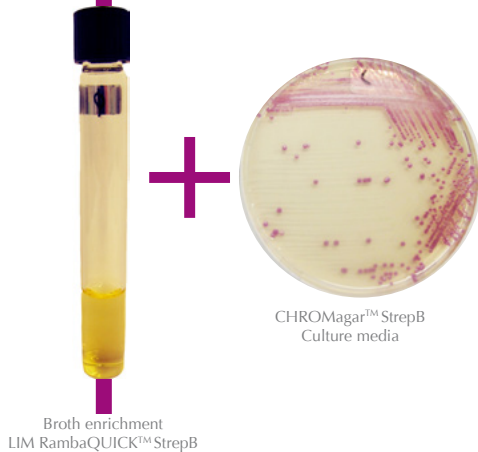
www.CHROMagar.com

For Group B Streptococcus (GBS) screening


Background

The CDC recommends pregnant women to be tested at 35 to 37 weeks of pregnancy for *Group B Streptococcal (GBS)* carriage by **selective enrichment broth of vaginal/anorectal swab samples**, followed by subculture. However, the selective broth formulas are not dedicated to the inhibition of *Enterococci*, usually part of the anorectal flora and able to hide the growth of GBS, source of false negative results.

LIM RambaQUICK™ StrepB is a selective enrichment broth **designed to largely reduce the growth of *Enterococci***, while allowing growth of GBS, and, therefore to raise the sensitivity of the prepartum screening method.

LIM RambaQUICK™ StrepB Method

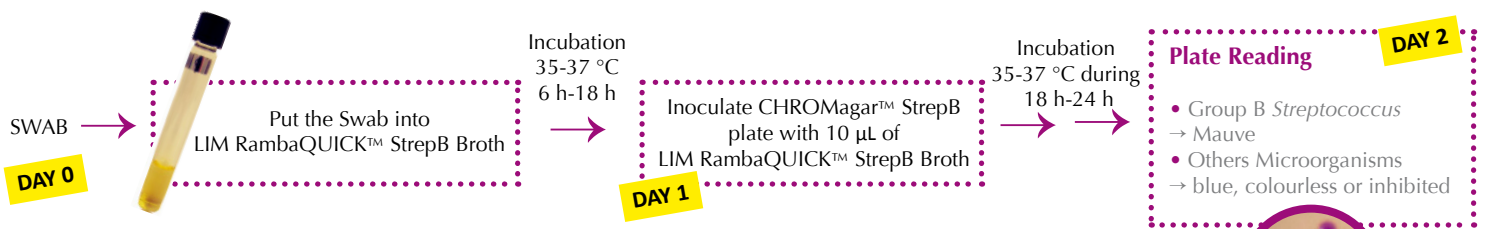Performance of the LIM RambaQUICK™ StrepB broth

Compared to classical LIM + subculture method:

INHIBITION OF MOST ENTEROCOCCI

LIM RambaQUICK™ StrepB is a selective enrichment broth **designed to largely reduce the growth of *Enterococci***, while allowing growth of GBS, and, therefore to raise the sensitivity of the prepartum screening method.

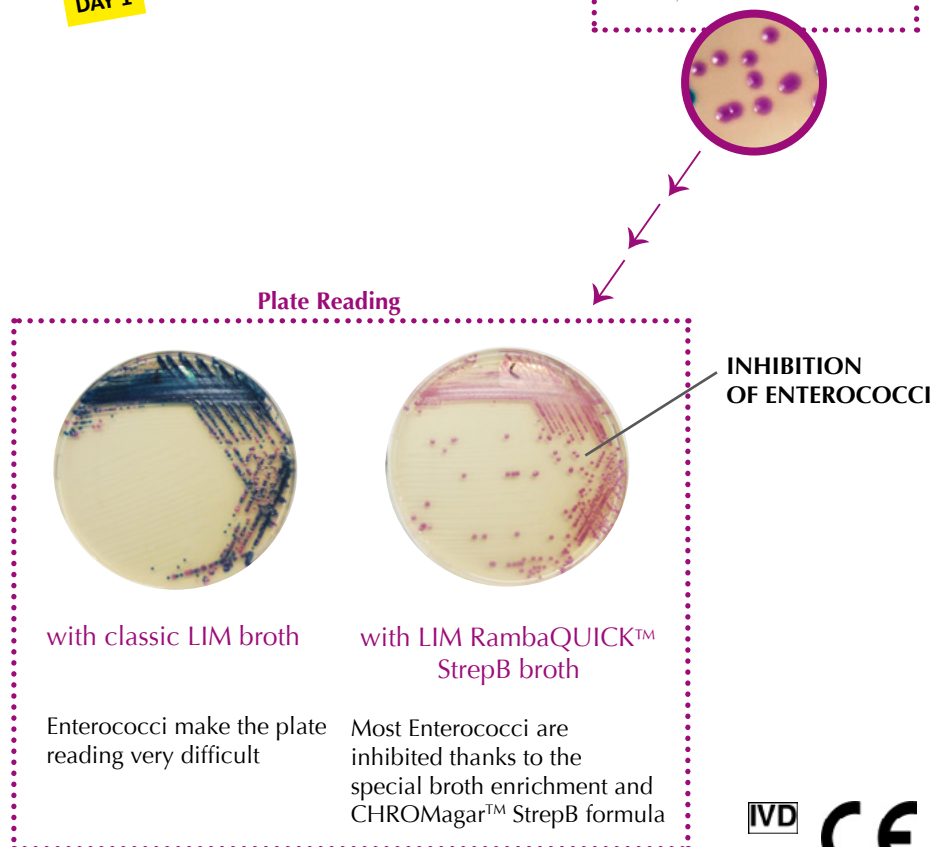
BROTH INOCULATION

Classical Method recommends overnight broth incubation for classical LIM. LIM RambaQUICK™ StrepB can be used also with overnight incubation, but has an optimal differentiation at 6h only as suggested in the Dr. Poisson Method Study.

Study: Prepartum vaginal/anorectal GroupB Streptococcus screening: Improvement of the enrichment step by the broth of additive RambaQUICK StrepB; D. Poisson et al. Centre Hospitalier Régional Orleans, France; Journal of Microbiology Methods 89 (2012) 107-109 doi:10.1016/j.mimet.2012.02.012

EASY INTERPRETATION OF THE SUBCULTURE PLATE

Easier reading of CHROMagar™ StrepB plate thanks to an intense mauve colony colouration.

Order References

Please use these product references when contacting your local distributor:

CHROMagar™ StrepB 5000 mL pack SB282
LIM RambaQUICK™ StrepB ... 5000 mL pack LB082

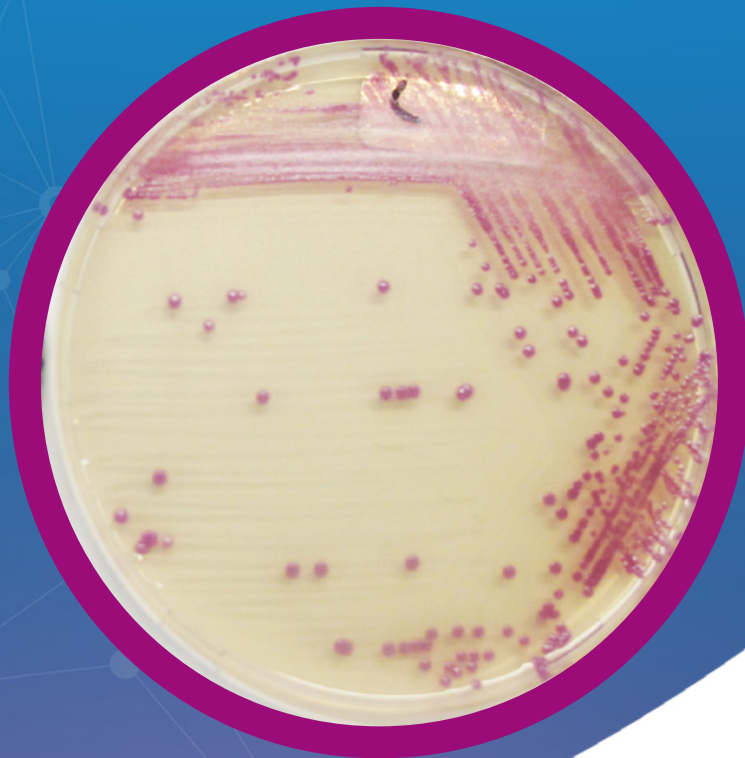
Manufacturer: CHROMagar, 29 avenue George Sand, 93210 La Plaine Saint-Denis - France
Email: CHROMagar@CHROMagar.com
Website: www.CHROMagar.com
Find your nearest distributor on www.CHROMagar.com/contact

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